TENOMAG

Art. TO1308MG

DESCRIZIONE / DESCRIPTION

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Medic-orthopedic support, functional and dynamic which extend and immobilize the rachis. The support, characterized by the frame in aluminium alloy, light adjustable and anatomic and the anatomic shoulder in elastic fabric, are studied for maintaining the rachis in a physiologic natural position, without compromise the normal respiration. The structure is composed by "Combined Resistex" material – Antistatic (the special conductive fibers used absorb and dissipate electrical charge), breathable and thermoregulator, antibacterial and hypo-allergenic, high performance (during physical exercise reduces the concentration of lactic acid improving blood circulation and cell oxygenation). The light weight and the elastic closing rods, guarantee a perfect anatomical adaptation. The system of adjustable passers in height allows the modulation of the dorso-lumbar push and the push on the shoulders. The posterior pocket for the insertion of dorsal rods allow the modulation of the intensity of support. Easy to fit and barely visible, guarantee optimal wearability.

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- 1 Put inside the magnetotherapy solenoid before wearing the brace
- 2 Wear the brace after placing solenoid in the right position.
- 3 Apply the device in standing position, better in supine position.
- 4 After choosing the size from the patient's measures, if necessary adjust the aluminium frame (or the dorsal sticks) in order to increase the anatomicity.
- 5 Enlarge the ties of the corset by temporarily fixing them on the anterior lateral oblique velcro areas (Fig.2 - A).
- 6 Insert the shoulder ties in the lateral passers, while taking care to make a double pass (Fig.1) first in the medial slot (A) and then in the lateral slot (B).
- 7 Temporarily fix the ends of the ties on the velcro area between the shoulder of the device and the rod (Fig.2 - B).
- 8 Wear the corset by inserting the arms in the shoulders of the device (Fig.2).
- 9 Fix the central velcro closing (Fig.3).
- 10 Tighten the corset by fixing the ties on the central anterior velcro, after removing them from the oblique lateral velcro (Fig.4).
- 11 If necessary adjust the height of the dorsal passers, by removing and placing them on the dorsal velcro (Fig.1).
- 12 Fix the ties of the shoulders on the central anterior velcro, after removing them from the velcro are (Fig.5-6).
- 13 If the ties of the shoulders are too long, remove the velcro ends, cut the excess and correctly replace them (Fig.7)

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Hand wash with warm water and mild soap, then rinse abundantly with clean water. Dry away from heat sources. Please refer to the seller in case of product concerns or components to be replaced. Do not dispose of in the environment.

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This device must be prescribed by a Physician and applied by a qualified Orthopaedic Technician who is the appropriate specialist to inform the wearer on product application and it's safe use. The product must be applied with the greatest of care in order to ensure it's efficacy in treatment. Any modification of any type must be decided by the Physician and undertaken by the Orthopaedic Technician. The product is recommended for single patient use. Direct contact of the product with delicate skin may cause irritation. Please contact a Physician immediately at the first signs of any intolerance.

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The product must not press on swellings, bruises or wounds. In case of product application doubts, please refer to a qualified Orthopaedic Technician. Do not wear this device in proximity of free flames.

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For the ORTHOMAG medical device functioning, within the TENOMAG box, we recommend to read the explanatory booklet inside the box.

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